EN Sterile safety lancet for single use

Intended purpose: Wellion HEEL STICK SAFETYLANCETS are sterile, single-use lancets that are used for the heel site puncture of premature babies (1-2,5kg) and newborns (2,5-9kg) to obtain capillary blood samples.

Intended users: Healthcare professionals.

Intended patient population: The intended patient population are premature babies (prenatal) and newborns (neonatal).

Article No.	Model / specification	Use
WELL2PRE-R	0,85 x 1,75 mm	Premature babies (prenatal) 1-2,5kg
WELL2NEO-R	1,00 x 2,50 mm	Newborns (neonatal) 2,5-9 kg

Indications: Examination of capillary blood in premature babies and newborns.

Clinical benefits: Use of Wellion HEEL STICK SAFETYLANCETS minimize procedure time of blood sampling, relieve pain and minimize rate of complication.

Contraindications: None

INSTRUCTION FOR USE

- Select the test site. Clean the site with an antiseptic swab and allow to air dry.
- Open the blister package and take out the lancet.
- Place the heel stick against the baby's heel and activate by pressing downward on the white latch. Then use the blood sample.
- Discard it into a biohazard sharps container.

Warnings: The protective blister keeps the lancet sterile. Do not use if protective blister is damaged or not on the lancet. | Heel sticks should only be performed on the lateral or medial plantar surface on the heel.* | Keep away from children. | Dispose according to local regulations. | For professional use only. | Serious incidents must be reported to the manufacturer and the competent authorities. * Following World Health Organization guidelines: Best Practices in Phlebotomy; 2010

Caution: If prenatal testing was unavailable, assume newborns of known or potential carriers may have genetic or clotting disorders (e.g. Hemophilia) and are at potential risk of excessive bleeding. Please avoid to perform heel pricks until results from coagulation tests are confirmed. | Do not perform heel sticks on newborns whose feet are edematous, injured, or infected or those who have anomalies that preclude putting pressure on the foot. | Do not perform heel sticks on areas that are bruised or injured by multiple previous heel sticks. For standard precautions follow World Health Organization publication „Standard precautions in health care“.

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© MED TRUST and Wellion are registered trademarks.